

Using healthPAL™ with the **AND** 321PBT Medical Scale



From the MedApps HealthPAL QuickStart Guide

Collecting Weight Measurements

1. Activate the scale by pressing the **Measurement Switch** gently. All display characters will be visible for several seconds.
2. Wait for the “O” symbol to appear with “0.0 lb”.
3. Step onto the scale and stand still while weight registers.
4. The scale will display your weight after which HealthPAL will visually display and audibly announce the current reading. (“**Your weight is XXX lbs.**”)

The measurement will be stored in HealthPAL’s memory and transmitted to server automatically when HealthPAL is inactive for 3 minutes.

Always follow manufacturer instructions for operation and safety for any accessory health monitors used with HealthPAL.

“Your Weight is: One-Hundred-Eighty Point Five Pounds.”

